## ① Revitalizing

Hyaluronic acid revitalization injectable solution 3 x5ml  
CE 0499

## FILLING AND MODELLING SOLUTION OF THE DERMAL MATRIX BASED ON HYALURONIC ACID

### Indications

Pluryal® Meso 1 is a sterile solution for intradermal use indicated for the prevention and treatment of skin damage.

### Instructions of use

Inject an appropriate amount of the Pluryal® Meso 1 solution into the surface layer of the dermis, in the following areas: glabella, cheekbones, nasolabial folds, bottom corner of the mouth and lower chin area.

### Duration of the treatment

Perform one session every 7 days, for a total of four sessions, then one session every 15 days for a total of two sessions. Follow-up: one session a month.

### Precautions for correct use

We strongly recommend obtaining the patient's prior informed consent to the treatment. Before applying the treatment, take the patient's medical history as accurately as possible. Explain the indications and the possible results of the treatment. Describe the contraindications and the possible undesired effects of the treatment, providing the necessary advice on the behaviour to be adopted and the rules followed immediately after the treatment. Observe the rules of asepsis and handling associated with this type of treatment. Avoid administering aspirin or vitamin C during the week prior to the treatment. Treatment during pregnancy and lactation is not recommended.

### Caution

This device is designed to be injected by a physician. The technique used is essential for the success of the treatment. Therefore this device must be used by doctors who have received specific training in the intradermal injection technique of filling solutions. Do not inject intravenously. Do not inject in areas, or in proximity of areas, affected by inflammatory or infectious skin conditions (e.g. acne, herpes, etc.). Each vial should be used for a single patient, to prevent the risk of cross-contamination. Do not use Pluryal® Meso 1 in patients with history of streptococcal or autoimmune disorders or undergoing immunological therapy. Do not use on patients liable to develop hypertrophic scars. Hyaluronic acid is incompatible with quaternary ammonium compounds, such as benzalconium chloride (50% creams or solutions). Pluryal® Meso 1 should never come into contact with these substances or with any medical / surgical instruments treated with these disinfectants. Disposable vials: once the vial has been opened, its content must be used immediately and any unused portion should be discarded together with the vial.

### Contraindications

The use of Pluryal® Meso 1 is contraindicated on patients who have shown hypersensitivity to any ingredients of the solution and should not be injected in the following cases:

- By means of injections other than intradermal injections.
- In association with peeling, laser or dermabrasion treatments. The doctor shall decide of the time that needs to elapse before the injection is made.
- Into any blood vessels.
- In children.

### Undesired effects

The following effects have been reported in the relevant literature:

- Inflammatory reactions, such as red rash, swelling or erythema, potentially associated with itching and / or pain around the point of injection, which generally tend to disappear in less than a week.
- Manifestation or lack of colour in the injection area.
- Cases of necrosis have also been reported in the glabellar region, abscesses, granulomas and hypersensitivity. The patient should be informed.

The patient should report to the doctor any of the previously described secondary effects that persist for more than a week, or the appearance of any other undesired effect.

### Ingredients

Hyaluronic acid sodium salt, L-isoleucine, L-leucine, L-lysine hydrochloride, L-proline, L-valine, Glycine, L-serine, L-alanine, L-cysteine, Sodium bicarbonate, water for injections.

### Storage

Store Pluryal® Meso 1 at 2-25°C (36-77°F) in a dry place in the original box. Protect from light, heat and frost and handle with care. The expiry date applies to the correctly stored product in the intact package.

### Content

Each box contains 3 ampoules of 5ml for intradermal use.

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